

RECOMMENDATION FOR SANITIZATION OF ROGER PRODUCTS IN CLASSROOMS



ROGER
TOUCHSCREEN MIC



ROGER INSPIRO MIC

These recommendations apply to the following Roger products when shared between users:

- Roger Touchscreen Mic
- Roger Inspiro Mic
- Roger Pass-around Mics

A surface disinfectant such as a solution with 70% isopropanol should be used to disinfect the Roger Mic.

When a student passes their Roger Mic to their teacher, the mic should be sanitized.

- The teacher should clean their hands with hand sanitizer prior to starting the sanitization process.
- Spray a piece of paper towel with the isopropanol disinfectant.
- Wipe the device off with the disposable paper towel.
- Never directly spray disinfectant directly onto the device.
- Make sure openings, in particular microphone openings, are never in contact with liquid as this may damage electronic components.
- While wiping the device with disinfectant, do not press any buttons as long as the device is still wet.
- Place Roger Transmitter around neck using the lanyard. Adjust lanyard so it is 6-8" from your mouth.
- Turn Roger Mic on.
- Clean hands with hand sanitizer after the sanitization process is complete.
- At the end of the class, repeat the sanitization process. Return the Roger Mic to the student.